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*User Manual - Manual de usuario - Benutzerhandbuch - Gebruiks Handleiding
Manual do Utilizador - Manuale d'istruzioni - Instrukcja obsługi - Használati
utasítás - Användarmanual - Ohjekirja - Brugermanual - Brukermanual -
Руководство пользователя - دليل المستخدم - מדרין למשתמש*

Fabricant / Manufacturer 

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Thank you !

We thank you for purchasing a Bluetens device.

Bluetens is a medical device. To make best use of it, you will find all relevant informations included in this user manual.

For any additional information or to simply give us your feedback, you can contact us at contact@bluetens.com.

Getting started with Bluetens is easy, here is some advice to let you benefit from it as quickly as possible and safely.

1. Please read the safety instructions as well as the contraindications at page 20. 
2. Your Smartphone should support the minimum system requirements :
iOS : iPhone 5s or above, iOS 7 or above
Android : Bluetooth 4.0 compatible and Android 6 or above
3. You will need to install the Bluetens App on your Smartphone.
 This application is available under « Bluetens Get Better »
 on Appstore & Google play.
4. Your Bluetooth must be ON.


It is not necessary to pair your Bluetens, your Smartphone will connect automatically as you will launch the program. This means that you will not need to fill in a PIN code or change your Bluetooth settings, Bluetens is small, strong and clever too!

Are all steps validated? So now we are ready to go through more serious things ;)

THE BLUETENS

- 1 ON/OFF Button
- 2 Intensity adjustment knob
- 3 Program intensity indicators
- 4 Charge indicator
- 5 Voltage indicator
- 6 Slider
- 7 Electrode cable plug
- 8 Micro USB plug
- 9 Reset button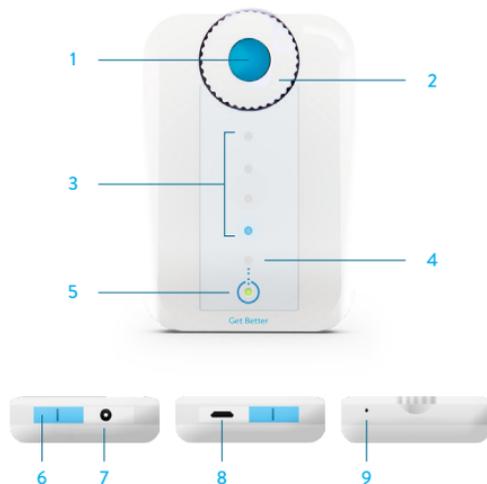

FIRST USE

LAUNCH THE BLUETENS APPLICATION


> Choose your product then create your account and read the tutorial.

> Choose from the three main functions on the home page :

ELECTROSTIMULATION SINGLE SESSION.

BASIC TREATMENT: a general guide on the different conditions

PREMIUM FUNCTION: so that one of our physios can help you to treat your problem.

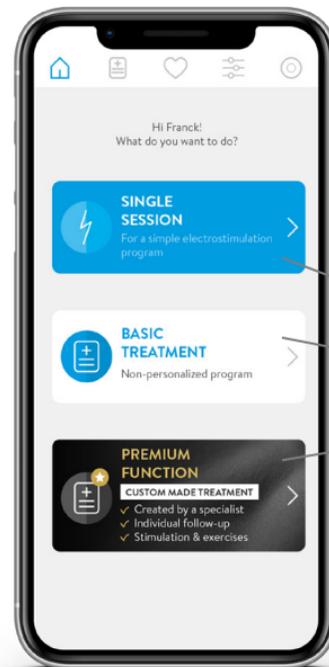ELECTROSTIMULATION SINGLE SESSION

> Choose the method and part of the body

+ CURE

For care providing fast pain relief

+ RELAX

For a relaxing massage or to ease muscle tension. It is also very effective to improve the quality and speed of recovery.


+ TONE

For a muscular work-out session.


> Then press «GO».

PROGRAM LAUNCH


PROGRAM DESCRIPTION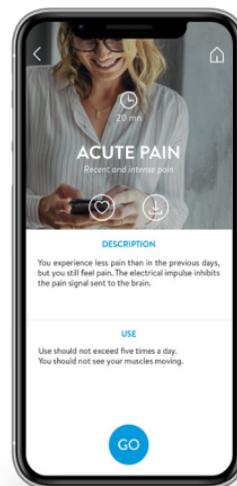


ELECTRODE PLACEMENT

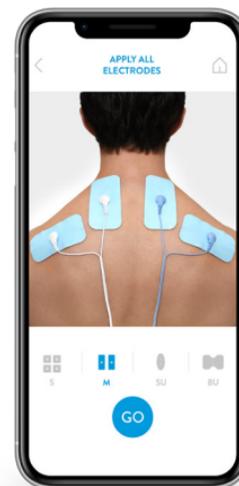TURN YOUR BLUETENS ON

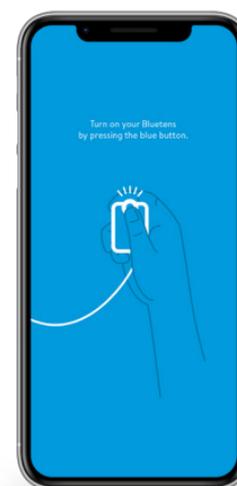RUNNING PROGRAM

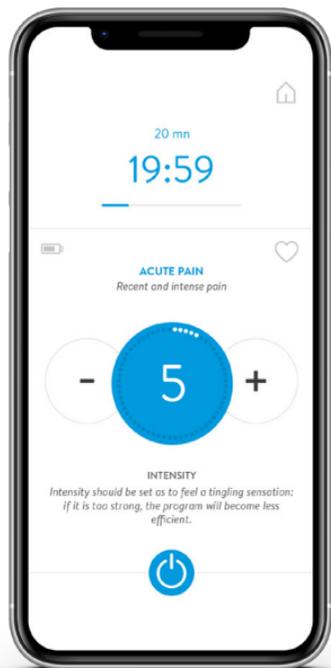DURING THE PROGRAM

> Adjust the program's intensity turning Bluetens wheel.

PROGRAM TERMINATION

> If you want to stop the program, press the «Stop» button. You will be redirected to the main menu.


END OF THE PROGRAM

> A message will indicate the end of the program. Turn Bluetens off before removing the electrodes. Reposition the electrodes on the good side of the plastic film and store them inside the sealed bag.

PREMIUM FUNCTION

PERSONALIZED SUPPORT TO REACH YOUR SPORT OR HEALTH GOALS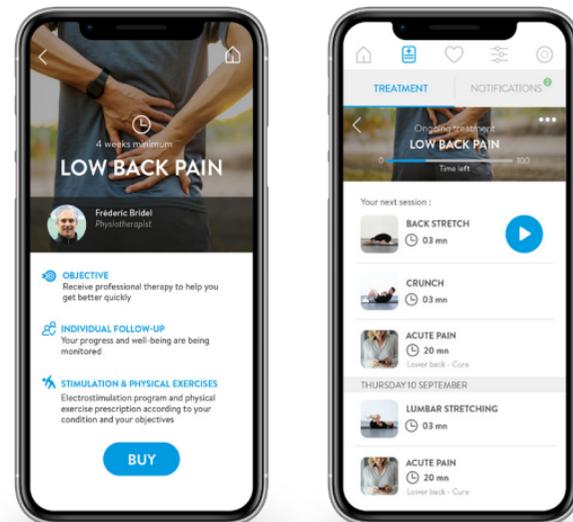

> Select your condition than complete your profile and proceed with payment.

Within 48 hours, you will receive your treatment and you should follow the instructions to get better quickly. The health professional will ensure that your exercises are carried out properly and that you feel good.

SAFETY INSTRUCTIONS & TECHNICAL CHARACTERISTICS

CONTRAINDICATIONS

The use of Bluetens is forbidden for people fitted with a pacemaker or epilepsy-prone. Bluetens use is also not recommended for pregnant women , and people with a hernia or an incisional hernia. People suffering from heart weakness must avoid areas around the heart. Don't use Bluetens if you have heart irregularities or heart diseases.

IMPORTANT !

Do not use Bluetens in water or in humid environments (sauna, balneotherapy and hammam, etc.). It is forbidden to place the electrodes on the sinocarotid region (in the front of the neck), on the transcranial region (through the head) or transthoraxale region (through the ribcage). To get it right, please follow the electrode placement's advices, they have been prepared by health professionals and correspond to areas of the body where Bluetens will be more effective.

- Weight: 25 grams
- Battery : 3.7V 400mAh
- Voltage : 0-70V (500Q load)
- Frequency : 1Hz-1200Hz
- Output Current : Maximum 300mA (pic), average of 20 mA
- Output voltage : Maximum 120 V (500Q load), 65V with load
- Power supply : USB
- USB (on charge) : 5V 500mA, 2.5W
- Impulse form : symmetrical biphasic square pulse
- Maximum resistance : 1500 Ohm

SAFETY PRECAUTIONS

Do not use Bluetens within a meter from shortwave medical devices or microwave equipment, which could modify the current generated by the device. Use only the cable provided by Bluetens. During sessions do not disconnect the electrode cable because power is on. The device must be turned off first. Never take off the electrodes when Bluetens is powered on. Never connect the electrode cable to an external power source. That could cause electric shock. Do not use any damaged or defective electrode cable. Always use the USB cable provided by Bluetens to charge the device. Never use Bluetens or the USB cable if damaged. Keep Bluetens and its accessories out of reach of children. Prevent any foreign matter (soil, water, metal, etc.) from entering the device. Do not use Bluetens while driving. Never recharge Bluetens during its use; a sliding part (the slider) prevents the simultaneity of recharge and use as an extra protection. In case of doubt, do not hesitate to consult a doctor.

- Automatic shut-off after each program : yes
- Ignition safety : 100%
- Classified as type BF 
- CE regulations concerning Medical Devices (Directive 93/42/EEC)  
- ROHS : devices built without materials that can have a harmful impact on the environment **RoHS**
- Protection against electric shock class 2 
- Device in contact with the patient out heart area 
- Read the label 

Information on waste management

At the end of the product life, make sure to discard it properly, following the relevant national rules. We thank you, in advance, for dropping the product in a place provided for this purpose in order to ensure a safe disposal in the respect of the environment.

RECOMMENDATIONS (storage and transport)

Temperature : 10°C / 45°C - Humidity level : 25% / 80%

Bluetens must be used only in accordance with the proposed programs. Bluetens Limited cannot be held responsible for any misuse or manipulation of Bluetens. Handle electrodes only when Bluetens is turned off. Bluetens is a medical device, do not leave it near children. Your Bluetens is a class 2A medical device certified by TÜV. It comes with a 2 year warranty. Hereby, Bluetens limited, declares that Bluetens is in compliance with the essential requirements and other relevant provisions of 1999/5/EC directive and 3/42/EEC directive 9. A copy of the Declaration of conformity is available on request to Bluetens Ltd, 2003 20/F Tower 5, China

Hong Kong City, 33 Canton Road, TST, Hong-Kong ou Bluetens France, 5 passage Saint Bernard, 75011 Paris, France and on

We manage all after-sale questions live.

If you need assistance after the purchase of your Bluetens from one of our distributors, do not return the product to the physical store, please contact us directly at contact@bluetens.com

Our agents will get back to you within 24 hours with a quick solution.

GUARANTEE

Bluetens is guaranteed for two years, this warranty does not cover damages caused by improper maintenance or usage not compliant with the related documentation, accidents (shock, fire, liquid), modifications, repairs, misuse or normal wear and tear. This guarantee covers material or manufacturing defects. This guarantee covers products purchased directly from Bluetens or from a Bluetens authorized dealer only.

GUARANTEE PERIOD

Your product is guaranteed two years from the date of purchase, the battery has a six month guarantee.

NOT COVERED BY THE GUARANTEE

Electrodes, clip, satchel, as well as cables and accessories included in the box do not benefit from this guarantee.

BLUETENS AT YOUR SERVICE

In the event of a defect covered by the guarantee, Bluetens will send you a return slip and will be responsible for the repairs or exchange of your device as well as of its return to you.

HOW TO GET CUSTOMER SUPPORT?

You can reach our Customer Service at any time at the following address:

contact@bluetens.com, a member of our team will be dedicated to find a solution to your request.

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User instructions for TENS

CAUTION

Do not use while sleeping.
Do not use while driving.
Do not continuously keep electrodes on body.
Do not use metal to touch the electrode.
Use this product with high frequency surgery devices might cause those device to become unstable.

MAIN DEVICE MAINTENANCE

- Clear regularly.
- Take off cables from the device and electrode after power off.
- Use soft cloth to clean if the device is dirty.
- Use wet cloth with neutral detergent or spirit to clean if necessary.
- Do not use spirit on the LED panel.
- Do not use other detergent.
- Keep water or liquids away from the device.

OPERATION INSTRUCTIONS

The device needs to be used together with the water proof carrying case in order to prevent any damage caused by water. Please put the device into the carrying bag during use .Then use the App to realize all settings and controlling of the device.

ELECTRODE PAD

Specification :

ITEM	STANDARD
Size	49.5±0.5 mm
Thickness	1.6 + 0.4 mm
Appearance	No stain, no gel spillover at the edges
Backing material	White non-woven cloth
Gel adhesion	Under normal humidity (60%), skin application is above 100 times after 72 hour unprotected storing
	Under high humidity (80%), skin application is above 30 times after 72-hour unprotected storing
	Under low humidity (20%), skin application is above 10 times after 72-hour unprotected storing
	Dried completely at 100°C for 1.5 hours, skin application is above 5 times
	For un-pretreated or sweaty skin, skin application is above 10 times.
	Comfortable cutaneous sensation
	No edge lift after 2-hour skin application under any condition
Skin adhesion reaction time: No need to press or press for less than 3 seconds	

ITEM	STANDARD
Conductivity	After 100 skin application, peel the gel off from the carbon film in the water and dry it. The impedance is <200ohms between any two points
	A.C. impedance for gel-to-metal by LCR meter: <100ohms at 100Hz; <80ohms at 1,000Hz <70ohms at 10,000Hz; <65ohms at 100,000Hz
Lead wire	Adaptable to male pin from 1.95~2.10m
	Strength between leadwire and connector is min. 4kg pulling force
	Strength between leadwire and electrode pad is min.3.kg pulling force
Adhesive force	Gel is strongly attached to the carbon film, hard to peel it off
Protective liner	110X110mm, 0.075mm transparent release liner, no gel residue when peel the electrode off
Printing	Lot number & expiry date: firm and legible
Safety	Biocompatibility of the gel G607 used for this product conforms to ISO10993-5-10
Package	120±5mmX185±5mm, transparent eco-friendly PE re-sealable bag
Applicable humidity	Humidity: 15%~70% Temperature: 0°C~30°C
Shelf life	2-year sealed storage under 30°C, gel color and adhesion remain unchanged

ELECTROMAGNETIC COMPATIBILITY DESCRIPTIONS

The EMC declaration according to the requirement of EN 60601-1-2

CAUTIONS

User must regard EMC, please install and put in service BLT15 according to the EMC information provided in the accompanying documents.
 Portable and mobile RF communications equipment can affect medical electrical equipment. The performance of the EQUIPMENT and SYSTEM that was determined to be essential performance.
 Table 201-Guidance and manufacturer's declaration-electromagnetic emissions-for BLT15, as following table.
 Table 202-Guidance and manufacturer's declaration-electromagnetic immunity -for BLT15, as following table.
 Table 204 -Guidance and manufacturer's declaration-electromagnetic immunity -for BLT15, as following table.
 Table 206 Recommended separation distances between portable and mobile RF communications equipment and BLT15, as following table.

Table 202

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC EMISSIONS		
The BLT15 is intended for use in the electromagnetic environment specified below. The customer or the user of the BLT15 should assure that it is used in such an environment.		
Immunity test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BLT15 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BLT15 is suitable for use in all establishments, including domestic establishments.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY

The BLT15 is intended for use in the electromagnetic environment specified below. The customer or the user of the BLT15 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± (2, 4, 6) kV contact ± (2, 4, 8) kV air	± (2, 4, 6) kV contact ± (2, 4, 8) kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT)	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the BLT15 requires continued operation during power mains interruptions, it is recommended that the BLT15 be powered from an uninterruptible power supply or a battery.
on power supply input lines IEC 61000-4-11	for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 sec		
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the test level

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY

The BLT15 is intended for use in the electromagnetic environment specified below.
The customer or the user of the BLT15 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the BLT15, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance 80 MHz to 800 MHz 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range's Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BLT15 is used exceeds the applicable RF compliance level above, the BLT15 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BLT15.

154 b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Table 206

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE BLT15

The BLT15 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BLT15 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BLT15 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2,5 GHz
	$d = 1,2\sqrt{P}$	$d = 1,2\sqrt{P}$	$d = 2,3\sqrt{P}$
0,01	0.12	0.12	0.23
0,1	0.38	0.38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

contact@bluetens.com
www.bluetens.com